



IC-1545-DL Multi-Flo DVT Combo

Intermittent Pneumatic Compression Device for DVT Prophylaxis

The IC-1545-DL Multi-FLO DVT Combo system has been designed to allow patients the freedom to provide DVT Prophylaxis as well as the reduction of acute post-operative edema in the comfort of their own homes.

Key Features:

- In-home use
- Adjustable Pressure
- Built in Compliance Meter allowing therapist and/or physician to monitor patient use
- Fully portable system
- Designed for easy use

The Multi-FLO DVT Combo offers three types of therapy:

Bilateral Calf

- Intended for prophylaxis of deep vein thrombosis

Bilateral Foot

- Intended for prophylaxis of deep vein thrombosis
- Enhancement of venous and arterial circulation
- Prevention of venous stasis ulcers
- Assist in healing of cutaneous ulcers

Post-Operative Edema (Shoulder, Ankle, Knee)

- Intended to help reduce pain and swelling after surgery
- Enhances faster healing and tissue regeneration required by the body



Specifications:

Dimensions: 6.25" h x 8" w x 8.25" d
Weight: 4.25 pounds

Cycle times

Bilateral calf mode (45-50 mmHg)
Inflation / hold: 13-17 seconds
Deflation / rest: 43-47 seconds
Cycle time: 60 second / 60 cycles per hour

Bilateral foot mode (90-120 mmhg)

Inflation / hold: 5-7 seconds
Deflation / rest: 53-55 seconds
Cycle time: 60 seconds / 60 cycles per hour

HCPCS - E0676

Prescription Required

PROUDLY MADE



IN THE USA





We at Bio Compression pride ourselves in manufacturing the highest quality of garments that are made in a variety of sizes and styles. We are the only manufacturer who produces custom garments within two weeks. All of our garments are Latex Free!

DVT Garments

Calf

- Our DVT Calf garment offers full compression around the entire leg
- Single Patient Use
- One Size Fits All



GID-3045-K
Knee High DVT sleeve

Foot

- Our DVT Foot garment provides compression on the entire foot
- Garment is very flexible and accommodates all foot sizes
- Easy to clean



GI-3045-F
Foot DVT Sleeve

Contraindications:

Use of this device is contraindicated for patients with the following condition:

- Any local condition of the extremity that would interfere with its application, including, but not limited to:
 - Dermatitis
 - Immediately following vein ligation
 - Gangrene
 - Skin grafts
 - Casts or splints
- Severe arteriosclerosis or other ischemic vascular disease
- Pulmonary edema
- Congestive heart failure
- Suspicion or confirmation of the presence of existing DVT
- Infections of the limb, including cellulitis without appropriate antibiotic coverage for at least 72 hours prior to treatment
- Presence of lymphangiosarcoma
- Inflammatory phlebitis or episodes of pulmonary embolism

Cryo Cold Therapy Garments

- Our Cryo Cold Therapy garments for this device can be purchased separately and come in knee, ankle and shoulder sleeves
- All Cryo Inserts are removable for more efficient freezer storage
- Custom Cryo Cold Therapy garments are available upon request



GC1-3035-SH
Cryo Shoulder



GC1-3045-K
Cryo Knee



GC1-3045-A
Cryo Ankle

PROUDLY MADE



IN THE USA

