



## SC-2004 Sequential Circulator

Model SC-2004 Sequential Circulator is a gradient, sequential, pneumatic compression device, intended for the primary or adjunctive treatment of primary or secondary Lymphedema. The device is additionally intended for alternate treatment of chronic venous stasis ulcers and associated venous insufficiency, as well as general treatment of swelling of the extremities. The SC-2004 is intended for home or hospital use.

### Key Features:

- Unilateral and/or Bilateral Functionality
- Non-paristaltic
- Custom Garment Availability
- Light Weight and Portable
- Easy to Use
- 3 Year Warranty

### Benefits:

- Reduces Pain
- Improves Lymphatic and Vascular Flow
- Promotes Wound Healing
- Prevents Infection
- Provides Effective Prophylaxis for Venous Thrombosis
- Reduces Dynamic Edema



SUPPLY ITEM: E0651



### Contraindications:

Pneumatic compression is contraindicated for patients with the following:

- Congestive heart failure;
- Deep Vein Thrombosis;
- Inflammatory phlebitis or episodes of pulmonary embolism;
- Infections in the limb, including cellulitis, without appropriate antibiotic coverage;
- Active Cancer except for palliative care

**WARNING:** Federal law restricts this device to sale by or on the order of a physician.



# Bio Compression Systems' Biocomfort Garments

Made of high quality 200 Denier Nylon Oxford coated with 3 mils of Polyurethane  
Available in a variety of sizes and styles including: over the shoulder,  
adjustable, custom and amputee.

## Upper Extremity/4-Chamber Garment Sizing Chart

Part Number	Size	Circumferences				Lengths		
		Axilla	Bicep	Elbow	Forearm	Wrist	Axilla to Fingertips	Shoulder to Fingertips
GS-3035-S	Small	23.5"	22"	19.5"	17.5"	15"	24"	26.5"
GS-3035-M	Medium	27"	26"	24"	21"	18"	28"	31"
GS-3035-L	Large	25.5"	24"	21.5"	19"	16"	31"	35.5"
GS-3035-SH/S	Small	23.5"	22"	19.5"	17.5"	15"	24"	26.5"+8"
GS-3035-SH/M	Medium	27"	26"	24"	21"	18"	28"	31"+8"
GS-3035-SH/L	Large	25.5"	24"	21.5"	19"	16"	31"	35.5"+8"

NOTE: Measurements above are in **Inches** and are the maximum allotted point on limb

## Lower Extremity/4-Chamber Garment Sizing Chart

Part Number	Size		Circumferences					Lengths		
			Thigh below Crotch	Mid Thigh	Mid Knee	Mid Calf	Ankle	Arch (Instep)	Crotch to Heel (Length)	Inguinal Crease
GS-3045-H	Half	Standard	N/A		29"	27"	23.5"	22"	19"	NA
GW-3045-H		Wide			30.5"	28"	29"	23"		
GA-3045-H		Adjustable			32"	30"	23.5"	22"		
GWA-3045-H		Adj. Wide			33.5"	31"	29"	23"		
GN-3045-S	Small	Narrow	25.5"	25"	22"	20"	18"	19.5"	25.5"	27"
GS-3045-S		Standard	31"	30"	28"	24.5"	21.5"	19.5"		
GW-3045-S		Wide	35.5"	34"	30.5"	28"	25.5"	21.5"		
GA-3045-S		Adjustable	34"	33"	31"	27.5"	21.5"	19.5"		
GWA-3045-S		Adj. Wide	38.5"	37"	33.5"	31"	25.5"	21.5"		
GN-3045-M	Medium	Narrow	27.5"	25.5"	23"	20"	18"	19.5"	28"	32"
GS-3045-M		Standard	32"	30"	28"	24.5"	21.5"	19.5"		
GW-3045-M		Wide	36.5"	34"	30"	28"	25.5"	22.5"		
GXW-3045		Extra Wide	44"	41"	36"	34"	30"	22.5"	29"	32"
GA-3045-M		Adjustable	35"	34"	31"	27.5"	21.5"	19.5"	28"	32"
GWA-3045-M		Adj. Wide	39.5"	36.5"	33.5"	31"	25.5"	22.5"		
GXWA-3045-M		Adj. Extra Wide	47"	44"	39"	37"	30"	22.5"	29"	32"
GN-3045-L	Large	Narrow	28"	26"	24"	21"	18"	19.5"	32"	36"
GS-3045-L		Standard	33"	32"	30"	26"	21.5"	19.5"		
GW-3045-L		Wide	37"	36"	32"	29"	24"	20.5"		
GA-3045-L		Adjustable	37"	36"	33"	29"	21.5"	19.5"		
GWA-3045-L		Adj. Wide	40"	38"	35"	32"	24"	22.5"		

NOTE: Measurements above are in **Inches** and are the maximum allotted point on limb



Bio Compression Systems  
120 West Commercial Avenue  
Moonachie, NJ 07074  
Phone: 800-888-0908/ Fax: 201-939-4503  
email: biosystems@biocompression.com  
www.biocompression.com