




EXTRICARE USA, LLC

extriCARE® 3600

Quick Start Guide

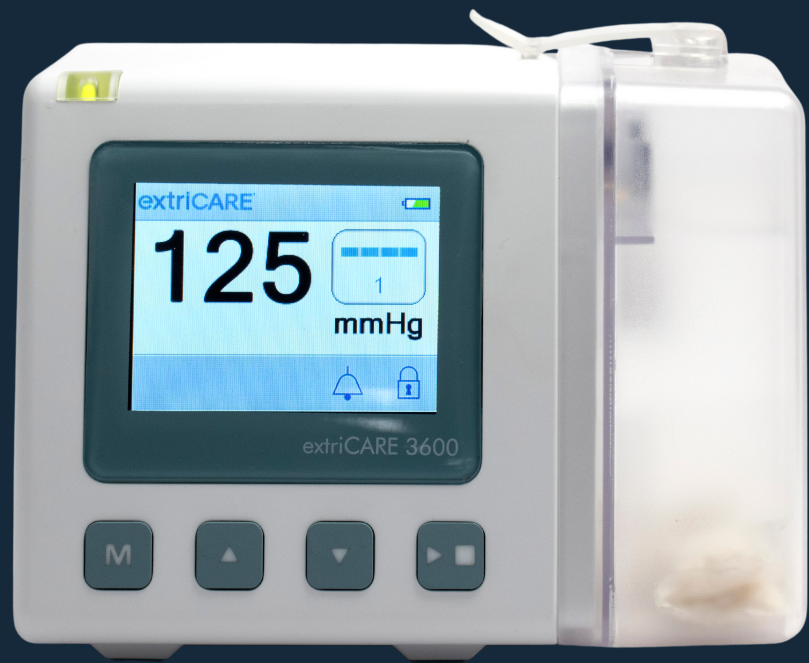

This Quick Guide does not replace the extriCARE 3600 Operating Manual. For information concerning intended use, indications, contraindications, warnings and cautions please refer to the Operating Manual available with the extriCARE 3600 pump. For customer service, email info@extricareusa.com or call 877.312.NPWT(6798).


Operating the Device

Canister Clip



1. **To Install Canister:** First, line up the 2 tabs on the bottom of the canister to the notches on the bottom of the extriCARE® 3600 device and then press canister clip into place.

Power Switch

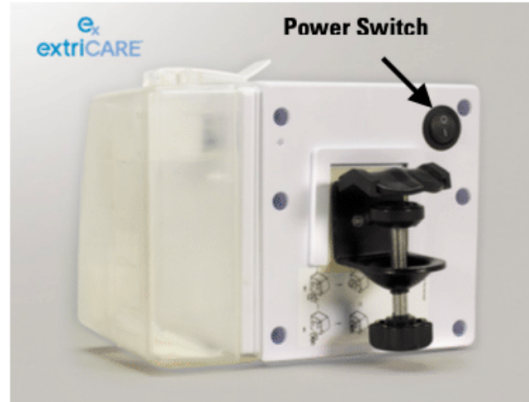

2. **POWER ON/OFF:** To power on the device, flip the POWER SWITCH located on the back panel of the device.



UP and DOWN

3. **Pressure Setting Controls:** To change the pressure, press the UP or DOWN button to set desired pressure setting. The pressure changes in increments of 5 mmHg with a range of 40 mmHg to 200 mmHg.

Mode Indicator



Mode Button

4. **Mode (M):** Press and Hold the Mode button until the unit beeps and the Mode indicator line changes. A solid line indicates "continuous" mode and a dashed line indicates "intermittent" mode. In continuous mode, the device maintains a continuous set pressure. In intermittent mode, the device will operate at the set pressure for 5 min, then decreases to 20 mmHg for 2 min and repeats this cycle.

Operating the Device (Cont'd)



Start/Stop Button

5. **Stop/Start Treatment:** To start or stop treatment, press and hold the START/STOP button for 2 seconds. **Please note you should hear a beep and a blue arrow indicator will become visible on the screen.*



Hold 2 sec. Unlock/Lock

6. **Lock/Unlock Device:** Device will lock automatically if there is no button input for 3 min. To unlock device, press and hold the MODE and START/STOP button for 2 sec. To manually lock the device, press and hold the MODE and START/STOP button for 2 sec.



7. **Rail Clamp:** To use the rail clamp, insert the clamp board firmly into the socket located at the back of the device (choose vertical for bed side clamp or horizontal for IV pole clamp). Un-screw the clamp (main black knob) to make space to secure clamp to pole (or bed). Screw clamp onto pole or bed. Make sure clamp is tight and pump is secure.



Charging Port

8. **Recharging Device:** To recharge device, use the Charger that is included with the extriCARE® 3600. Charging port is located on the left side of the device. To ensure charging, turn on the pump and make sure power adapter icon is displayed.

extriCARE® 3600 Error/Troubleshooting




Canister Installation Error

Cause/Description: The canister is not detected or is installed incorrectly.
System Status: Pump remains on.
Suggested Mitigation: Properly install the canister.



High Voltage Error

Cause/Description: Pump is being used with an adapter that is not recommended; There is a risk of voltage incompatibility if the input voltage is greater than 16V.
System Status: Fuse may blowout.
Suggested Mitigation: Unplug the adapter and use the recommended adapter.

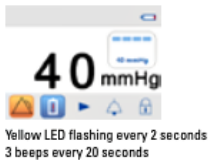


Low Battery Error

Cause/Description: When the battery contains less than 10% power. This indicates that the system will shut off soon.
System Status: Pump remains functioning until the battery depletes completely.
Suggested Mitigation: Plug the unit in allowing it to function and charge simultaneously.



Canister Full Error

Cause/Description: The canister is equipped with full sensors that will be triggered either when the canister is full of exudates, or a false fullness is caused by incorrect use of the system.
System Status: Pump will shut off immediately.
Suggested Mitigation: Install a new canister.

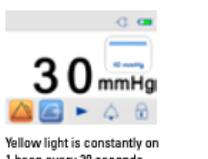


Minor Leakage

Cause/Description: Pump is unable to reach 80% of the preset pressure after 5 minutes of pump effort.
System Status: Pump remains on.
Suggest Mitigation: Inspect for possible air leaks between the wound and dressing, the dressing and canister, and the pump and canister. If necessary, power off and back on to restart the system after adjustment.



Severe Leakage

Causes/Description: Pump unable to reach 50% of the preset pressure after 2 minutes of pump effort.
System Status: Pump remains on but will shut down after 10 minutes of continuous alarm and without any operation.
Suggested Mitigation: Follow same steps as Minor Leakage Error.

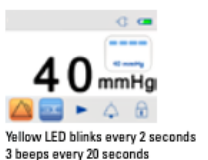

Blockage

Cause/Description: Tubing or dressing clog or blockage.
System Status: Pump remains on.
Suggested Mitigation: Replace with new dressing and tubing set.



EXTRICARE USA, LLC

*This concludes the extriCARE® 3600 pump quick start guide.
If you have any questions, please call 877.312.6798
(toll-free) or email info@extricareusa.com.*



SCAN ME



QUICK START VIDEO GUIDE

Address:

2460 General Armistead Ave, Suite 205
Norristown, PA 19403

Phone: 877-312-NPWT (6798)

Email: info@extricareusa.com

Website: www.extricareusa.com